



ASSUT EUROPE
SUTURE CHIRURGICHE



More than surgery

Medifuser:®



ELASTOMERIC PUMP



MEDIFUSER® is a device for continuous infusion therapy employing elastomeric technology. It consists of a balloon made of elastic material (elastomer) contained in an enclosure made of rigid plastic material. The fluid contained in the reservoir is subjected to a constant pressure that pushes it along an infusion line, without the need for electrical power supply. It is indicated for different applications, from anesthetic therapies to antibiotic therapies, from chemotherapy to Patient Controlled Analgesia (PCA).

COMFORTABLE AND EASY TO USE

The elastomeric pump fills up very easily and its use can be easily explained both to the HCPs and to the patient or his family members. Moreover, being very handy and not bulky, improves the quality of life of the patient. It doesn't require the use of electrical power or additional equipment and it can be used both in home therapies and hospitalized patients.

SAFE AND RELIABLE

Medifuser® maintains a predefined constant flow in relation to the patient's status and the level of medication required, eliminating any risk of overdose. The elastomer does not cause the alteration of drugs. The presence of the anti-particles filter guarantees the absence of infections or contaminations.

Medifuser® is available in a wide range of models that include both the continuous administration of the medication and the possibility to infuse the bolus at fixed intervals.

CODES

CONTINUOUS FLOW INFUSION

CODE	VERSION	DESCRIPTION	BALLOON VOLUME	FLOW RATE	BOX
P050C20	Transparent	Continuous flow infusion, filter 5 micron	50 ml	2 ml/hr	80 pcs
P060C05	Transparent	Continuous flow infusion, filter 5 micron	60 ml	0,5 ml/hr	80 pcs
P060C05B	Brown				
P060C20	Transparent	Continuous flow infusion, filter 5 micron	60 ml	2 ml/hr	80 pcs
P060C20B	Brown				
P100C05	Transparent	Continuous flow infusion, filter 5 micron	100 ml	0,5 ml/hr	80 pcs
P100C05B	Brown				
P100C20	Transparent	Continuous flow infusion, filter 5 micron	100 ml	2 ml/hr	80 pcs
P100C20B	Brown				
P275C20	Transparent	Continuous flow infusion, filter 5 micron	275 ml	2 ml/hr	60 pcs
P275C20B	Brown				
P275C50	Transparent	Continuous flow infusion, filter 5 micron	275 ml	5 ml/hr	60 pcs
P275C50B	Brown				
P300C50	Transparent	Continuous flow infusion, filter 5 micron	300 ml	5 ml/hr	60 pcs
P400C50	Transparent	Continuous flow infusion, filter 5 micron	400 ml	5 ml/hr	60 pcs

MULTIRATE INFUSION

CODE	VERSION	DESCRIPTION	BALLOON VOLUME	FLOW RATE	BOX
P060C20M	Transparent	Continuous flow infusion, filter 5 micron	60 ml	0/2/4/6/8/10/12/14 ml/h	80 pcs
P060C20MB	Brown				
P100C20M	Transparent	Continuous flow infusion, filter 5 micron	100 ml	0/2/4/6/8/10/12/14 ml/h	80 pcs
P100C20MB	Brown				
P275C20M	Transparent	Continuous flow infusion, filter 5 micron	275 ml	0/2/4/6/8/10/12/14 ml/h	60 pcs
P275C20MB	Brown				

P.C.A. INFUSION (PATIENT CONTROLLED ANALGESIA) WITH BOLUS

CODE	VERSION	DESCRIPTION	BALLOON VOLUME	FLOW RATE	BOX
P060P20	Transparent	PCA Infusion with bolus 0.5ml/15 min, filter 5 micron	60 ml	2 ml/hr	80 pcs
P060P20B	Brown				
P100P20	Transparent	PCA Infusion with bolus 0.5ml/15 min, filter 5 micron	100 ml	2 ml/hr	80 pcs
P100P20B	Brown				
P275P20	Transparent	PCA Infusion with bolus 0.5ml/15 min, filter 5 micron	275 ml	2 ml/hr	60 pcs
P275P20B	Brown				

MULTIRATE INFUSION + P.C.A. INFUSION (PATIENT CONTROLLED ANALGESIA) WITH BOLUS

CODE	VERSION	DESCRIPTION	BALLOON VOLUME	FLOW RATE	BOX
P060P20M	Transparent	Multirate Infusion + PCA with bolus 0.5ml/15 min, filter 5 micron	60 ml	0/2/4/6/8/10/12/14 ml/h	80 pcs
P060P20MB	Brown				
P100P20M	Transparent	Multirate Infusion + PCA with bolus 0.5ml/15 min, filter 5 micron	100 ml	0/2/4/6/8/10/12/14 ml/h	80 pcs
P100P20MB	Brown				
P275P20M	Transparent	Multirate Infusion + PCA with bolus 0.5ml/15 min, filter 5 micron	275 ml	0/2/4/6/8/10/12/14 ml/h	60 pcs
P275P20MB	Brown				

ITALY

Headquarter

Zona Industriale 67062
Magliano dei Marsi (AQ), Italy
Tel. +39 0863 517956/515000
Fax +39 0863 570084/515209

Legal Office

Via Giuseppe Gregoraci, 12
00173 Rome
Tel. +39 06 72677348
Fax +39 06 72675380



FRANCE

86-114 Avenue Louis Roche
Batiment D Hall-3 92230 Gennevilliers
Tel. +33 147985294
Fax +33 147985892

ESPAÑA

Poligono Ind. Mutilva Baja
Calle V, n.16 31192 - Mutilva Baja
Tel. +34 948122274
Fax +34 948120842

UK

Unit 12° Barncliffe Mills Shelley HD8 8LU
Huddersfield
Tel. +44 1484602222
Fax +44 1484602280

USA

55 NE 5th Avenue,
Suite 501
33432 - 4093 Boca Raton FL
United States

RUSSIAN FEDERATION

c/o Studio Marzona
Rozhdestvenskiy Bulvar n. 10/7 - 107031
Moscow
Tel. +7 962 9862804

BRAZIL

Rua Professor Alfredo Gomes n. 14
CEP 22551-080
Botafogo Rio de Janeiro (RJ)
Tel. 0055 21 22661943
Fax 0055 21 22663539

ALGERIE

Z.A. Dar El Beida, Lot 66 BT 12
Alger

TUNISIE

Rue 7316 Villa N°5- EL Menzah 9B
1013 Tunis-Tunisie
Tel. 00216 71875740
Fax 00216 71880541

GERMANY

Prager Strasse 34 04317
Leipcsig

MAROC

Résidence Azhar - 24
Rue Jalal Eddine Essayauti
Rez De Chaussée 2 - 20100
Casablanca

www.assuteurope.com
info@assuteurope.com

