



ASSUT EUROPE

SUTURE CHIRURGICHE




®

More than surgery


Antigen COMBO

Rapid Test

SARS-CoV-2/Influenza A+B

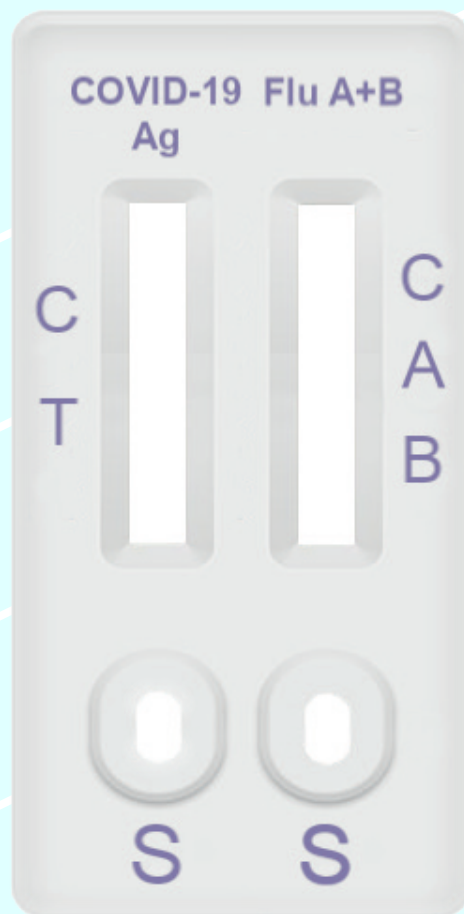

The SARS-CoV-2/Influenza A+B Antigen COMBO Rapid Test Cassette is a lateral flow immunoassay intended for the qualitative detection of SARS-CoV-2, influenza A and influenza B viral nucleoprotein antigens in nasopharyngeal swab and from individuals suspected of respiratory viral infection consistent with COVID-19 by their healthcare provider. Symptoms of respiratory viral infection due to SARS-CoV-2 and influenza can be similar.

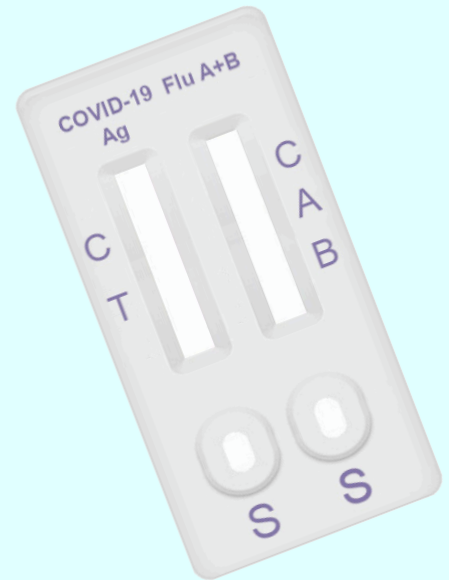

Kit Contents

Extraction reagents



Test Cassette



Work Station




Extraction Tube & Dropper Tip



Sterilized Swab



Sample Preparation and Test Procedure

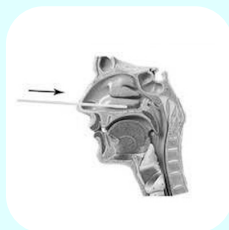

1. Insert sterilized swab through the nostril parallel to the palate.




2. Gently rub and roll the swab, and leave swab in place for several seconds to absorb secretions.




3. Slowly remove swab while rotating it.




4. Add all of the specimen extraction reagent into an extraction tube, and put it on the work station.

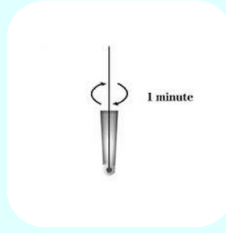

5. Insert the swab into the extraction tube. Roll the swab at least 5 times while pressing the head against the bottom and side of the extraction tube, and stay for 1 minute.

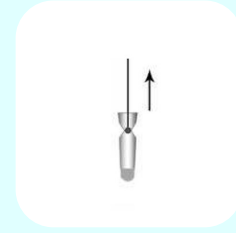


6. Pinch the extraction tube and remove the swab.

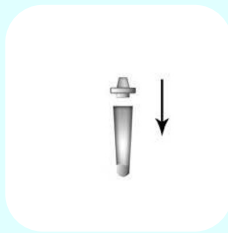

7. Insert the dropper tip into the tube.





8. Transfer 3 drops to the specimen well of the test cassette.

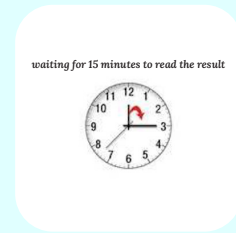

9. Interpret the test results at 15 minutes. Do not read results after 20 minutes.

Interpretation of Results

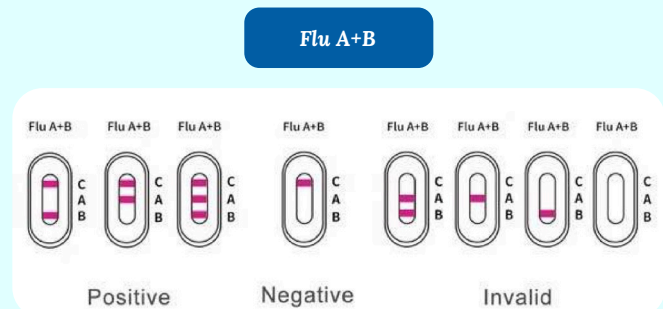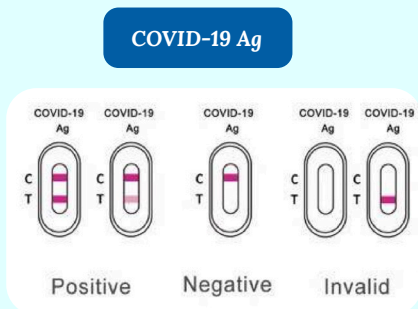

PRODUCT CODE	DESCRIPTION	BOX
INF-COV/25	<i>Antigen COMBO Rapid Test SARS-CoV-2/Influenza A+B</i>	25



ASSUT EUROPE
SUTURE CHIRURGICHE



More than surgery

ITALIA

Stabilimento di Produzione

Zona Industriale
67062 Magliano dei Marsi (AQ), Italy
Tel. +39 0863 517956/515000
Fax +39 0863 570084/515209

Ufficio Legale

Via G. Gregoraci, n.12
00173 Roma
Tel. +39 06 72677348
Fax +39 06 72675380

FRANCE

86-114 Avenue Louis Roche
Batiment D Hall-3 92230
Gennevilliers
Tel. +33 147985294
Fax +33 147985892

ESPAÑA

Poligono Landaben
Calle C, nave 1y2 - 31012
Pamplona (Navarra)
Tel. +34 948122274
Fax. +34 948120842

GERMANY

Prager Strasse 34
04317 Leipzig

UK

Unit 12a Barncliffe Mills
Shelley HD8 8LU Huddersfield
Tel. +44 1484602222



USA

55 NE 5th Avenue, Suite 501
33432 - 4093 Boca Raton FL
United States

RUSSIAN FEDERATION

c/o Studio Marzona
Rozhdestvenskiy B-r 10 Bld. 7
107031 Moscow
Tel/Fax +7 962 9862804

BRASIL

Rua Professor Alfredo Gomes, 14
CEP 22551-080
Botafogo Rio De Janeiro (RJ)
Tel.: 0055 21 22661943
Fax: 0055 21 22663539

ALGERIE

Zone D'Activité Dar El Beida
Lot n° 66, Bt 12
Alger

TUNISIE

Rue 7316 Villa n°5 - El Menzah 9B
1013 Tunis, Tunisie
Tel.: 00216 71875740
Fax: 00216 71880541

www.assuteurope.com

info@assuteurope.com



©2020 Assut Europe S.p.A. All rights reserved
CE 0373 - UNI EN ISO 9001:2015
UNI EN ISO 13485:2016
UNI EN ISO 14001:2015